



Synpla Nano Facial Protocol

Pre-Treatment

- It is recommended for the client to use Pigment Corrector Gel at least 2 weeks prior to treatment.

Treatment

1. Cleanse the skin using Mega Rich Cleansing Gel.
2. Second cleanse using the Mandelic Cleanser.
3. Tone using the Everyday Renewing Toner.
4. (Optional) Apply Extraction in a Fraction to the T-Zone, and any other area that may need to be extracted manually. Wait 3-5 minutes.
5. Tone using Everyday Renewing Toner to refresh the skin and remove any residue from extractions or extraction solution.
6. Draw 1ML of M.A.D Platinum SYNPLA Anti Wrinkle Growth Factor Serum or SYNPLA Bright Growth Factor Serum into your syringe.
7. Attach a nano tip to a covered NanoStamp 360 device.
8. Apply the Selected M.A.D Platinum SYNPLA Serum to the area you are going to treat and using a gliding motion, nano infuse over the serum for enhanced penetration. Repeat in all areas of the face.
9. Massage in any remaining serum.
10. Apply Pigment Corrector Gel, Universal Hyrda Moisturizer, and Hypersheer SPF 50.



M.A.D Products Needed

- Mega Rich Cleansing Gel
- Mandelic Cleanser
- Everyday Renewing Toner
- Extraction in a Fraction
- M.A.D Platinum SYNPLA Growth Factor Serum or SYNPLA Bright Serum M.A.D Platinum Hyaluronic Acid Hydrating Facial Mask
- Hypersheer SPF 50

NanoStamp 360 Products Needed

- NanoStamp 360 device
- Protective Barrier Sheath
- Nano tip
- Pigment Corrector Gel
- Universal Hydra Moisturizer